



Ebrington
M E D I C A L

1A Carrakeel Drive
Maydown Industrial Estate
MAYDOWN BT47 6UQ
Tel: 028 7134 5188
Email: info@ebrington-ni.com

CALIBRATION & FUNCTIONAL CHECKING

TEST RECORDS AND REPORTS

<i>Practice/Surgery</i>
X
X
X
X
X
X

<i>Date of Testing:</i>	X
-------------------------	---

<i>Technician:</i>	X
--------------------	---



COMMENTS & RECOMMENDATIONS

The surgery's equipment was tested under normal operating conditions using calibrated test meters and units. Equipment was found to be in good condition generally, with the exceptions noted below.

Requiring immediate attention:

- Replacement of otoscope set with faulty handle
- Replacement of defibrillator pads
- Replacement of fingerclip pulse oximeter
- Replacement of faulty hand held sphygmomanometers as noted

We would also recommend consideration be given to:

- Replacing cuffs as noted

Please review the general notes on the equipment test certificates for additional comments specific to each item.

EQUIPMENT PASS RATE = 89%

TEST SUMMARY

X X X X X 0	
	Tested on X
	Tested by X

Ref	Item	Model	Serial Number	Location / Item Nr	Pass/Fail	Notes and Comments
1	Digital Floor Scales	Seca 875	5875301176197	Room 1	Pass	
2	Digital Floor Scales	Seca 875	362182	Room 2	Pass	
3	Digital Column Scales	Seca 704	5704341168531	Room 3	Pass	
4	Digital Floor Scales	Seca 875	373717	Room 4	Pass	
5	Digital Floor Scales	Seca 875	839638	Room 5	Pass	
6	Digital Floor Scales	Seca 875	5875164166804	Room 6	Pass	
7	Digital Floor Scales	Seca 875	373719	Room 7	Pass	
8	Digital Floor Scales	Seca 875	839594	Room 8	Pass	
9	Otoscope/Ophthalmoscope Set	Heine EN100		Room 1	Pass	
10	Otoscope/Ophthalmoscope Set	Heine EN100		Room 2	Pass	
11	Otoscope/Ophthalmoscope Set	Heine EN100		Room 3	Pass	
12	Otoscope/Ophthalmoscope Set	Heine EN100		Room 4	Pass	
13	Otoscope/Ophthalmoscope Set	Heine EN100		Room 5	Pass	
14	Otoscope/Ophthalmoscope Set	Opticlar		Room 6	Pass	
15	Otoscope/Ophthalmoscope Set	Heine EN100		Room 7	Pass	
16	Otoscope/Ophthalmoscope Set	Heine EN100		Room 8	Pass	

<i>Ref</i>	<i>Item</i>	<i>Model</i>	<i>Serial Number</i>	<i>Location / Item Nr</i>	<i>Pass/Fail</i>	<i>Notes and Comments</i>
17	Otoscope/Ophthalmoscope Set	Heine Mini 3000		Room 8	Pass	
18	Otoscope/Ophthalmoscope Set	Welch Allyn Elite H/H		Eamon	Pass	
19	Otoscope/Ophthalmoscope Set	Keeler H/H		Eamon	Fail	Handle faulty
20	Infra-Red Thermometer	Thermoflash	104251203549	Room 2	Pass	
21	Infra-Red Thermometer	DT-8836		Room 2	Pass	
22	Infra-Red Thermometer	Panodyne	G1783	Emergency Trolley	Pass	
23	Ear Thermometer	Genius 3	N19831178	Emergency Trolley	Pass	
24	Ear Thermometer	Genius 3	N20727237	Emergency Trolley	Pass	
25	Blood Pressure Monitor	Omron HEM-907	20101200041AF	Room 1	Pass	
26	Blood Pressure Monitor	Welch Allyn ProBP 2400		Room 2	Pass	
27	Blood Pressure Monitor	Omron HEM-907	20101200042AF	Room 3	Pass	HEM-907 cuff recommended
28	Blood Pressure Monitor	Omron HEM-907	20101200044AF	Room 4	Pass	
29	Blood Pressure Monitor	Omron HEM-907	20110800403AF	Room 5	Pass	
30	Blood Pressure Monitor	Welch Allyn ProBP 2400		Room 6	Pass	
31	Blood Pressure Monitor	Omron HEM-907	20101200043AF	Room 8	Pass	
32	Blood Pressure Monitor	Omron M3	20170437007VG	Emergency Trolley	Pass	
33	Nebuliser	Nebulair	18A4730108	Emergency Trolley	Pass	
34	Defibrillator	Zoll AED+	X07A104909	Emergency Trolley	Fail	Pads expired
35	24Hr Ambulatory BP Monitor	Mobil-O-Graph	C13609	Room 1	Pass	
36	24Hr Ambulatory BP Monitor	Mobil-O-Graph	C25573	Room 1	Pass	
37	Doppler	Huntleigh D900 8MHz	29792-08	Room 1	Pass	
38	Spirometer	Microlab	51546		Pass	
39	Calibration Syringe	Carefusion 3 Litre	95389670		Pass	

TEST CERTIFICATE

Aneroid Sphygmomanometer

X X X X X X	Test Number	
	Tested on	X
	Tested by	X

<i>Item</i>	Aneroid Sphygmomanometer
<i>Manufacturer</i>	
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Blood Pressure Monitor

X X X X X X	Test Number	
	Tested on	X
	Tested by	X

<i>Item</i>	Blood Pressure Monitor
<i>Manufacturer</i>	
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature





1A Carrakeel Drive
 Maydown Industrial Estate
 MAYDOWN BT47 6UQ
 Tel: 028 7134 5188
 Email: info@ebrington-ni.com

TEST CERTIFICATE

Pulse Oximeter

X X X X X X	Test Number	
	Tested on	X
	Tested by	X

<i>Item</i>	Pulse Oximeter
<i>Manufacturer</i>	
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result
%SpO2	Pass
HR (BPM)	Pass
Visual	Pass
Function	Pass
	+0.00

General Notes

Pass / Fail Pass

Signature




1A Carrakeel Drive
 Maydown Industrial Estate
 MAYDOWN BT47 6UQ
 Tel: 028 7134 5188
 Email: info@ebrington-ni.com

TEST CERTIFICATE

Auroscope / Otoscope

X X X X X X	Test Number	
	Tested on	X
	Tested by	X

<i>Item</i>	Auroscope / Otoscope
<i>Manufacturer</i>	
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	
<i>Tolerance</i>	
<i>Location/Item Nr</i>	


Test	Result	Comment
Visual	Pass	
Function	Pass	
Clean Lens	Pass	
Bulb Test	Pass	

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Digital Medical Scales

X X X X X X	Test Number	
	Tested on	X
	Tested by	X

<i>Item</i>	Digital Medical Scales		
<i>Manufacturer</i>			
<i>Model</i>			
<i>Serial Number</i>			
<i>Type</i>			
<i>Tolerance</i>	± 0.2kg	<i>Range</i>	0 - 200kgs
<i>Location/Item Nr</i>			

Calibrated Load (kgs)	Tolerance ±0.2kg	Result
20	19.80kg - 20.20kg	Pass
40	39.80kg - 40.20kg	Pass
60	59.80kg - 60.20kg	Pass
80	79.80kg - 80.20kg	Pass
90	89.80kg - 90.20kg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Mechanical Medical Scales

X X X X X X	Test Number	
	Tested on	X
	Tested by	X

<i>Item</i>	Mechanical Medical Scales		
<i>Manufacturer</i>			
<i>Model</i>			
<i>Serial Number</i>			
<i>Type</i>			
<i>Tolerance</i>	± 1.0kg	<i>Range</i>	0 - 200kgs
<i>Location/Item Nr</i>			

Calibrated Load (kgs)	Tolerance ±1.0kg	Result
20	19.00kg - 21.00kg	Pass
40	39.00kg - 41.00kg	Pass
60	59.00kg - 61.00kg	Pass
80	79.00kg - 81.00kg	Pass
90	89.00kg - 91.00kg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Medical Baby Scales

X X X X X X	Test Number	
	Tested on	X
	Tested by	X

<i>Item</i>	Medical Baby Scales		
<i>Manufacturer</i>			
<i>Model</i>			
<i>Serial Number</i>			
<i>Type</i>			
<i>Tolerance</i>	± 1.0%	<i>Range</i>	0 - 15kgs
<i>Location/Item Nr</i>			

<i>Calibrated Load</i>	<i>Scales Reading</i>	<i>Differential</i>
1		-1.00
5		-5.00
10		-10.00
		+0.00
		+0.00

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Nebuliser

X X X X X X	Test Number	
	Tested on	X
	Tested by	X

<i>Item</i>	Nebuliser
<i>Manufacturer</i>	
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	
<i>Tolerance</i>	>1 bar >7ltr/min
<i>Location/Item Nr</i>	

Test	Result	Differential
Visual	Pass	
Function	Pass	
Flow Rate	l/min	Fail
Pressure	Pass	
		+0.00

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Spirometer

X X X X X X	Test Number	
	Tested on	X
	Tested by	X

<i>Item</i>	Spirometer
<i>Manufacturer</i>	
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	
<i>Tolerance</i>	± 3%
<i>Location/Item Nr</i>	

Test	Result
Visual	Pass
Function	Pass
Calibration	Pass
Expiatory	
Inspiratory	

General Notes

Pass / Fail

Pass

Signature





1A Carrakeel Drive
 Maydown Industrial Estate
 MAYDOWN BT47 6UQ
 Tel: 028 7134 5188
 Email: info@ebrington-ni.com

TEST CERTIFICATE

Doppler

X X X X X X	Test Number	
	Tested on	X
	Tested by	X

<i>Item</i>	Doppler
<i>Manufacturer</i>	
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result
Visual	Pass
Function	Pass
Interference	Pass

General Notes

Pass / Fail Pass

Signature


TEST CERTIFICATE

Thermometer

X X X X X X	<i>Test Number</i>	
	<i>Tested on</i>	X
	<i>Tested by</i>	X

<i>Item</i>	Thermometer
<i>Manufacturer</i>	
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

<i>Test</i>	<i>Result</i>	<i>Comments</i>
Test Read	Pass	
Visual	Pass	
Function	Pass	
		+0.00
		+0.00

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

ECG

X X X X X X	Test Number	
	Tested on	X
	Tested by	X

<i>Item</i>	ECG
<i>Manufacturer</i>	
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	
<i>Tolerance</i>	


Test	Result
NSR @ 60 BPM	Pass
NSR @ 120 BPM	Pass
Print	Pass
Battery	Pass
Simulator	Pass

General Notes

Pass / Fail

Pass

Signature





TEST CERTIFICATE

0

X X X X X X	<i>Test Number</i>	
	<i>Tested on</i>	X
	<i>Tested by</i>	X

<i>Item</i>	
<i>Manufacturer</i>	
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

<i>Test</i>	<i>Result</i>
Visual	Pass
Function	Pass

General Notes

Pass / Fail

Pass

Signature

Aneroid Sphygmomanometer	Pass
Auroscope / Otoscope	Fail
Baby Scales	
Blood Pressure Monitor	YES
Defibrillator	NO
Doppler	
Ear Syringe	
Ear Thermometer	
ECG Machine	
Electrosurgery Device	
Handheld Ultrasound	
IR Thermometer	
LED Examination Light	
Medical Scales	
Nebuliser	
Ophthalmoscope	
Patient Couch	
Pulse Oximeter	
Sphygmomanometer	
Spirometer	
Vaccine Fridge	

Audiometer

Accosson
A&D Medical
Boso
Braun
Carefusion
Cozaar
Edan
Keeler
Heartstart
Heine
Huntleigh
Istin
Laerdal
Lec Medical
Medix
Microlife
Mobilograph
Nihon Kohden
Nonin
Omron

Propulse
Riester
Roche
Seca
Tycos
Welch Allyn
Zoll

Hand held
Desk mounted
Mobile
Wall mounted

Mechanical floor scales
Digital floor scales
Mechanical column scales
Digital column scales
Baby scales
Platform scale

Desk mounted
Portable

Hand held
Fingerclip type
Desk mounted

Hand held
Desk mounted

Ear thermometer
IR thermometer
Probe type

TEST CERTIFICATE

Digital Medical Scales

X X X X X X	Test Number	1
	Tested on	X
	Tested by	X

<i>Item</i>	Digital Medical Scales		
<i>Manufacturer</i>	Seca		
<i>Model</i>	875		
<i>Serial Number</i>	5875301176197		
<i>Type</i>	Digital floor scales		
<i>Tolerance</i>	± 0.2kg	<i>Range</i>	0 - 200kgs
<i>Location/Item Nr</i>			

Calibrated Load (kgs)	Tolerance ±0.2kg	Result
20	19.80kg - 20.20kg	Pass
40	39.80kg - 40.20kg	Pass
60	59.80kg - 60.20kg	Pass
80	79.80kg - 80.20kg	Pass
90	89.80kg - 90.20kg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Digital Medical Scales

X X X X X X	Test Number	2
	Tested on	X
	Tested by	X

<i>Item</i>	Digital Medical Scales		
<i>Manufacturer</i>	Seca		
<i>Model</i>	875		
<i>Serial Number</i>	362182		
<i>Type</i>	Digital floor scales		
<i>Tolerance</i>	± 0.2kg	<i>Range</i>	0 - 200kgs
<i>Location/Item Nr</i>			

Calibrated Load (kgs)	Tolerance ±0.2kg	Result
20	19.80kg - 20.20kg	Pass
40	39.80kg - 40.20kg	Pass
60	59.80kg - 60.20kg	Pass
80	79.80kg - 80.20kg	Pass
90	89.80kg - 90.20kg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Digital Medical Scales

X X X X X X	Test Number	3
	Tested on	X
	Tested by	X

<i>Item</i>	Digital Medical Scales		
<i>Manufacturer</i>	Seca		
<i>Model</i>	704		
<i>Serial Number</i>	5704341168531		
<i>Type</i>	Digital column scales		
<i>Tolerance</i>	± 0.2kg	<i>Range</i>	0 - 200kgs
<i>Location/Item Nr</i>			

Calibrated Load (kgs)	Tolerance ±0.2kg	Result
20	19.80kg - 20.20kg	Pass
40	39.80kg - 40.20kg	Pass
60	59.80kg - 60.20kg	Pass
80	79.80kg - 80.20kg	Pass
90	89.80kg - 90.20kg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Digital Medical Scales

X X X X X X	Test Number	4
	Tested on	X
	Tested by	X

<i>Item</i>	Digital Medical Scales		
<i>Manufacturer</i>	Seca		
<i>Model</i>	875		
<i>Serial Number</i>	373717		
<i>Type</i>	Digital floor scales		
<i>Tolerance</i>	± 0.2kg	<i>Range</i>	0 - 200kgs
<i>Location/Item Nr</i>			

Calibrated Load (kgs)	Tolerance ±0.2kg	Result
20	19.80kg - 20.20kg	Pass
40	39.80kg - 40.20kg	Pass
60	59.80kg - 60.20kg	Pass
80	79.80kg - 80.20kg	Pass
90	89.80kg - 90.20kg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Digital Medical Scales

X X X X X X	Test Number	5
	Tested on	X
	Tested by	X

<i>Item</i>	Digital Medical Scales		
<i>Manufacturer</i>	Seca		
<i>Model</i>	875		
<i>Serial Number</i>	839638		
<i>Type</i>	Digital floor scales		
<i>Tolerance</i>	± 0.2kg	<i>Range</i>	0 - 200kgs
<i>Location/Item Nr</i>			

Calibrated Load (kgs)	Tolerance ±0.2kg	Result
20	19.80kg - 20.20kg	Pass
40	39.80kg - 40.20kg	Pass
60	59.80kg - 60.20kg	Pass
80	79.80kg - 80.20kg	Pass
90	89.80kg - 90.20kg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Digital Medical Scales

X X X X X X	Test Number	6
	Tested on	X
	Tested by	X

<i>Item</i>	Digital Medical Scales		
<i>Manufacturer</i>	Seca		
<i>Model</i>	875		
<i>Serial Number</i>	5875164166804		
<i>Type</i>	Digital floor scales		
<i>Tolerance</i>	± 0.2kg	<i>Range</i>	0 - 200kgs
<i>Location/Item Nr</i>			

Calibrated Load (kgs)	Tolerance ±0.2kg	Result
20	19.80kg - 20.20kg	Pass
40	39.80kg - 40.20kg	Pass
60	59.80kg - 60.20kg	Pass
80	79.80kg - 80.20kg	Pass
90	89.80kg - 90.20kg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Digital Medical Scales

X X X X X X	Test Number	7
	Tested on	X
	Tested by	X

<i>Item</i>	Digital Medical Scales		
<i>Manufacturer</i>	Seca		
<i>Model</i>	875		
<i>Serial Number</i>	373719		
<i>Type</i>	Digital floor scales		
<i>Tolerance</i>	± 0.2kg	<i>Range</i>	0 - 200kgs
<i>Location/Item Nr</i>			

Calibrated Load (kgs)	Tolerance ±0.2kg	Result
20	19.80kg - 20.20kg	Pass
40	39.80kg - 40.20kg	Pass
60	59.80kg - 60.20kg	Pass
80	79.80kg - 80.20kg	Pass
90	89.80kg - 90.20kg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Digital Medical Scales

X X X X X X	Test Number	8
	Tested on	X
	Tested by	X

<i>Item</i>	Digital Medical Scales		
<i>Manufacturer</i>	Seca		
<i>Model</i>	875		
<i>Serial Number</i>	839594		
<i>Type</i>	Digital floor scales		
<i>Tolerance</i>	± 0.2kg	<i>Range</i>	0 - 200kgs
<i>Location/Item Nr</i>			

Calibrated Load (kgs)	Tolerance ±0.2kg	Result
20	19.80kg - 20.20kg	Pass
40	39.80kg - 40.20kg	Pass
60	59.80kg - 60.20kg	Pass
80	79.80kg - 80.20kg	Pass
90	89.80kg - 90.20kg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Auroscope / Otoscope

X X X X X X	Test Number	9
	Tested on	X
	Tested by	X

<i>Item</i>	Auroscope / Otoscope
<i>Manufacturer</i>	Heine
<i>Model</i>	EN100
<i>Serial Number</i>	
<i>Type</i>	Wallmounted
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comment
Visual	Pass	
Function	Pass	
Clean Lens	Pass	
Bulb Test	Pass	

General Notes

Pass / Fail

Pass

Signature





1A Carrakeel Drive
 Maydown Industrial Estate
 MAYDOWN BT47 6UQ
 Tel: 028 7134 5188
 Email: info@ebrington-ni.com

TEST CERTIFICATE

Auroscope / Otoscope

X X X X X X	Test Number	10
	Tested on	X
	Tested by	X

<i>Item</i>	Auroscope / Otoscope
<i>Manufacturer</i>	Heine
<i>Model</i>	EN100
<i>Serial Number</i>	
<i>Type</i>	Wallmounted
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comment
Visual	Pass	
Function	Pass	
Clean Lens	Pass	
Bulb Test	Pass	

General Notes

Pass / Fail

Pass

Signature

TEST CERTIFICATE

Auroscope / Otoscope

X X X X X X	Test Number	11
	Tested on	X
	Tested by	X

<i>Item</i>	Auroscope / Otoscope
<i>Manufacturer</i>	Heine
<i>Model</i>	EN100
<i>Serial Number</i>	
<i>Type</i>	Wallmounted
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comment
Visual	Pass	
Function	Pass	
Clean Lens	Pass	
Bulb Test	Pass	

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Auroscope / Otoscope

X X X X X X	Test Number	12
	Tested on	X
	Tested by	X

<i>Item</i>	Auroscope / Otoscope
<i>Manufacturer</i>	Heine
<i>Model</i>	EN100
<i>Serial Number</i>	
<i>Type</i>	Wallmounted
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comment
Visual	Pass	
Function	Pass	
Clean Lens	Pass	
Bulb Test	Pass	

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Auroscope / Otoscope

X X X X X X	Test Number	13
	Tested on	X
	Tested by	X

<i>Item</i>	Auroscope / Otoscope
<i>Manufacturer</i>	Heine
<i>Model</i>	EN100
<i>Serial Number</i>	
<i>Type</i>	Wallmounted
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comment
Visual	Pass	
Function	Pass	
Clean Lens	Pass	
Bulb Test	Pass	

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Auroscope / Otoscope

X X X X X X	Test Number	14
	Tested on	X
	Tested by	X

<i>Item</i>	Auroscope / Otoscope
<i>Manufacturer</i>	Opticlar
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	Wallmounted
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comment
Visual	Pass	
Function	Pass	
Clean Lens	Pass	
Bulb Test	Pass	

General Notes

Pass / Fail

Pass

Signature





1A Carrakeel Drive
 Maydown Industrial Estate
 MAYDOWN BT47 6UQ
 Tel: 028 7134 5188
 Email: info@ebrington-ni.com

TEST CERTIFICATE

Auroscope / Otoscope

X X X X X X	Test Number	15
	Tested on	X
	Tested by	X

<i>Item</i>	Auroscope / Otoscope
<i>Manufacturer</i>	Heine
<i>Model</i>	EN100
<i>Serial Number</i>	
<i>Type</i>	Wallmounted
<i>Tolerance</i>	
<i>Location/Item Nr</i>	


Test	Result	Comment
Visual	Pass	
Function	Pass	
Clean Lens	Pass	
Bulb Test	Pass	

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Auroscope / Otoscope

X X X X X X	Test Number	16
	Tested on	X
	Tested by	X

<i>Item</i>	Auroscope / Otoscope
<i>Manufacturer</i>	Heine
<i>Model</i>	EN100
<i>Serial Number</i>	
<i>Type</i>	Wallmounted
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comment
Visual	Pass	
Function	Pass	
Clean Lens	Pass	
Bulb Test	Pass	

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Auroscope / Otoscope

X X X X X X	Test Number	17
	Tested on	X
	Tested by	X

<i>Item</i>	Auroscope / Otoscope
<i>Manufacturer</i>	Heine
<i>Model</i>	Mini 3000
<i>Serial Number</i>	
<i>Type</i>	Hand held
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comment
Visual	Pass	
Function	Pass	
Clean Lens	Pass	
Bulb Test	Pass	

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Auroscope / Otoscope

X X X X X X	Test Number	18
	Tested on	X
	Tested by	X

<i>Item</i>	Auroscope / Otoscope
<i>Manufacturer</i>	Welch Allyn
<i>Model</i>	Elite
<i>Serial Number</i>	
<i>Type</i>	Hand held
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comment
Visual	Pass	
Function	Pass	
Clean Lens	Pass	
Bulb Test	Pass	

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Auroscope / Otoscope


X X X X X X	Test Number	19
	Tested on	X
	Tested by	X

<i>Item</i>	Auroscope / Otoscope
<i>Manufacturer</i>	Keeler
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	Hand held
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comment
Visual	Fail	
Function	Fail	
Clean Lens	Fail	
Bulb Test	Fail	

General Notes Handle faulty so unable to test.

Pass / Fail **Fail**

Signature 

TEST CERTIFICATE

Thermometer

X X X X X X	Test Number	20
	Tested on	X
	Tested by	X

<i>Item</i>	Thermometer
<i>Manufacturer</i>	Thermoflash
<i>Model</i>	LX-316T
<i>Serial Number</i>	104251203549
<i>Type</i>	Hand held IR Thermometer
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comments
Test Read	Pass	
Visual	Pass	
Function	Pass	
		+0.00
		+0.00

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Thermometer

X X X X X X	Test Number	21
	Tested on	X
	Tested by	X

<i>Item</i>	Thermometer
<i>Manufacturer</i>	
<i>Model</i>	DT-8836
<i>Serial Number</i>	
<i>Type</i>	Hand held IR Thermometer
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comments
Test Read	Pass	
Visual	Pass	
Function	Pass	
		+0.00
		+0.00

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Thermometer

X X X X X X	Test Number	22
	Tested on	X
	Tested by	X

<i>Item</i>	Thermometer
<i>Manufacturer</i>	Panodyne
<i>Model</i>	
<i>Serial Number</i>	G1783
<i>Type</i>	Hand held IR Thermometer
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comments
Test Read	Pass	
Visual	Pass	
Function	Pass	
		+0.00
		+0.00

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Thermometer

X X X X X X	Test Number	23
	Tested on	X
	Tested by	X

<i>Item</i>	Thermometer
<i>Manufacturer</i>	Covidien
<i>Model</i>	Genius 3
<i>Serial Number</i>	N19831178
<i>Type</i>	Ear Thermometer
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comments
Test Read	Pass	
Visual	Pass	
Function	Pass	
		+0.00
		+0.00

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Thermometer

X X X X X X	Test Number	24
	Tested on	X
	Tested by	X

<i>Item</i>	Thermometer
<i>Manufacturer</i>	Covidien
<i>Model</i>	Genius 3
<i>Serial Number</i>	N20727237
<i>Type</i>	Ear Thermometer
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Comments
Test Read	Pass	
Visual	Pass	
Function	Pass	
		+0.00
		+0.00

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Blood Pressure Monitor

X X X X X X	Test Number	25
	Tested on	X
	Tested by	X

<i>Item</i>	Blood Pressure Monitor
<i>Manufacturer</i>	Omron
<i>Model</i>	HEM-907
<i>Serial Number</i>	20101200041AF
<i>Type</i>	Wallmounted
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Blood Pressure Monitor

X X X X X X	Test Number	26
	Tested on	X
	Tested by	X

<i>Item</i>	Blood Pressure Monitor
<i>Manufacturer</i>	Welch Allyn
<i>Model</i>	ProBP 2400
<i>Serial Number</i>	
<i>Type</i>	Wallmounted
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Blood Pressure Monitor

X X X X X X	Test Number	27
	Tested on	X
	Tested by	X

<i>Item</i>	Blood Pressure Monitor
<i>Manufacturer</i>	Omron
<i>Model</i>	HEM-907
<i>Serial Number</i>	20101200042AF
<i>Type</i>	Wallmounted
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Blood Pressure Monitor

X X X X X X	Test Number	28
	Tested on	X
	Tested by	X

<i>Item</i>	Blood Pressure Monitor
<i>Manufacturer</i>	Omron
<i>Model</i>	HEM-907
<i>Serial Number</i>	20101200044AF
<i>Type</i>	Wallmounted
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Blood Pressure Monitor

X X X X X X	Test Number	29
	Tested on	X
	Tested by	X

<i>Item</i>	Blood Pressure Monitor
<i>Manufacturer</i>	Omron
<i>Model</i>	HEM-907
<i>Serial Number</i>	20110800403AF
<i>Type</i>	Wallmounted
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Blood Pressure Monitor

X X X X X X	Test Number	30
	Tested on	X
	Tested by	X

<i>Item</i>	Blood Pressure Monitor
<i>Manufacturer</i>	Welch Allyn
<i>Model</i>	ProBP 2400
<i>Serial Number</i>	
<i>Type</i>	Wallmounted
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Blood Pressure Monitor

X X X X X X	Test Number	31
	Tested on	X
	Tested by	X

<i>Item</i>	Blood Pressure Monitor
<i>Manufacturer</i>	Omron
<i>Model</i>	HEM-907
<i>Serial Number</i>	20101200043AF
<i>Type</i>	Wallmounted
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Blood Pressure Monitor

X X X X X X	Test Number	32
	Tested on	X
	Tested by	X

<i>Item</i>	Blood Pressure Monitor
<i>Manufacturer</i>	Omron
<i>Model</i>	M3
<i>Serial Number</i>	20170437007VG
<i>Type</i>	Desktop
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Nebuliser

X X X X X X	Test Number	33
	Tested on	X
	Tested by	X

<i>Item</i>	Nebuliser
<i>Manufacturer</i>	Flaem
<i>Model</i>	Nebulair
<i>Serial Number</i>	
<i>Type</i>	Portable
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	Differential
Visual	Pass	
Function	Pass	
Flow Rate	12.00 l/min	Pass
Pressure	Pass	
		+0.00

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Defibrillator

X X X X X X	Test Number	34
	Tested on	X
	Tested by	X

<i>Item</i>	Defibrillator
<i>Manufacturer</i>	Zoll
<i>Model</i>	AED+
<i>Serial Number</i>	X07A104909
<i>Type</i>	Portable
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result
Visual	Pass
Function	Pass
Batteries	Pass
Pads	Fail

General Notes Pads expired

Pass / Fail **Fail**

Signature 

TEST CERTIFICATE

Blood Pressure Monitor

X X X X X X	Test Number	35
	Tested on	X
	Tested by	X

<i>Item</i>	Blood Pressure Monitor
<i>Manufacturer</i>	IEM
<i>Model</i>	Mobil-O-Graph
<i>Serial Number</i>	C13609
<i>Type</i>	Ambulatory 24hr BPM
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Blood Pressure Monitor

X X X X X X	Test Number	36
	Tested on	X
	Tested by	X

<i>Item</i>	Blood Pressure Monitor
<i>Manufacturer</i>	IEM
<i>Model</i>	Mobil-O-Graph
<i>Serial Number</i>	C25573
<i>Type</i>	Ambulatory 24hr BPM
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Doppler

X X X X X X	Test Number	37
	Tested on	X
	Tested by	X

<i>Item</i>	Doppler
<i>Manufacturer</i>	Huntleigh
<i>Model</i>	D900
<i>Serial Number</i>	29792-08
<i>Type</i>	Hand held
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result
Visual	Pass
Function	Pass
Interference	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Spirometer

X X X X X X	Test Number	38
	Tested on	X
	Tested by	X

<i>Item</i>	Spirometer
<i>Manufacturer</i>	Carefusion
<i>Model</i>	Microlab
<i>Serial Number</i>	51546
<i>Type</i>	Desktop
<i>Tolerance</i>	± 3%
<i>Location/Item Nr</i>	

Test	Result
Visual	Pass
Function	Pass
Calibration	Pass
Expiatory	Pass
Inspiratory	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Calibration Syringe

X X X X X X	Test Number	39
	Tested on	X
	Tested by	X

<i>Item</i>	Calibration Syringe
<i>Manufacturer</i>	Carefusion
<i>Model</i>	3 Litre
<i>Serial Number</i>	95389670
<i>Type</i>	Desk mounted
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result
Visual	Pass
Function	Pass
Calibration Test	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Aneroid Sphygmomanometer

X X X X X X	Test Number	40
	Tested on	X
	Tested by	X

<i>Item</i>	Aneroid Sphygmomanometer
<i>Manufacturer</i>	Welch Allyn
<i>Model</i>	Presstabil
<i>Serial Number</i>	08033612
<i>Type</i>	Hand held
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Aneroid Sphygmomanometer

X X X X X X	Test Number	41
	Tested on	X
	Tested by	X

<i>Item</i>	Aneroid Sphygmomanometer
<i>Manufacturer</i>	Heine
<i>Model</i>	Gamma XXL LF
<i>Serial Number</i>	500000981
<i>Type</i>	Wall mounted
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Aneroid Sphygmomanometer

X X X X X X	Test Number	42
	Tested on	X
	Tested by	X

<i>Item</i>	Aneroid Sphygmomanometer
<i>Manufacturer</i>	Timesco
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	Hand held
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Aneroid Sphygmomanometer

X X X X X X	Test Number	43
	Tested on	X
	Tested by	X

<i>Item</i>	Aneroid Sphygmomanometer
<i>Manufacturer</i>	
<i>Model</i>	
<i>Serial Number</i>	
<i>Type</i>	Hand held
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Aneroid Sphygmomanometer

X X X X X X	Test Number	44
	Tested on	X
	Tested by	X

<i>Item</i>	Aneroid Sphygmomanometer
<i>Manufacturer</i>	Heine
<i>Model</i>	Gamma XXL LF
<i>Serial Number</i>	500001028
<i>Type</i>	Wall mounted
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Pass
250	247mmHg - 253mmHg	Pass
200	197mmHg - 203mmHg	Pass
150	147mmHg - 153mmHg	Pass
100	97mmHg - 103mmHg	Pass

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Aneroid Sphygmomanometer

X X X X X X	Test Number	45
	Tested on	X
	Tested by	X

<i>Item</i>	Aneroid Sphygmomanometer
<i>Manufacturer</i>	Welch Allyn
<i>Model</i>	DS54
<i>Serial Number</i>	130725192144
<i>Type</i>	Hand held
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Fail
250	247mmHg - 253mmHg	Fail
200	197mmHg - 203mmHg	Fail
150	147mmHg - 153mmHg	Fail
100	97mmHg - 103mmHg	Fail

General Notes Out of tolerance. Reading high in all bands.

Pass / Fail **Fail**

Signature



TEST CERTIFICATE

Aneroid Sphygmomanometer


X X X X X X	Test Number	46
	Tested on	X
	Tested by	X

<i>Item</i>	Aneroid Sphygmomanometer
<i>Manufacturer</i>	Welch Allyn
<i>Model</i>	DS54
<i>Serial Number</i>	18011073749
<i>Type</i>	Hand held
<i>Tolerance</i>	± 3mmHg
<i>Location/Item Nr</i>	

Sphyg Reading	Acceptable Pressure Range	Within Tolerance Range?
280	277mmHg - 283mmHg	Fail
250	247mmHg - 253mmHg	Fail
200	197mmHg - 203mmHg	Fail
150	147mmHg - 153mmHg	Fail
100	97mmHg - 103mmHg	Fail

General Notes Needle mispositioned. Unable to test.

Pass / Fail **Fail**

Signature 

TEST CERTIFICATE

Pulse Oximeter

X X X X X X	Test Number	47
	Tested on	X
	Tested by	X

<i>Item</i>	Pulse Oximeter
<i>Manufacturer</i>	Anapulse
<i>Model</i>	ANP100
<i>Serial Number</i>	
<i>Type</i>	Fingerclip
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result	
%SpO2	Pass	
HR (BPM)	Pass	
Visual	Pass	
Function	Pass	
		+0.00

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Pulse Oximeter

X X X X X X	Test Number	48
	Tested on	X
	Tested by	X

<i>Item</i>	Pulse Oximeter
<i>Manufacturer</i>	Anapulse
<i>Model</i>	ANP100
<i>Serial Number</i>	
<i>Type</i>	Fingerclip
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result
%SpO2	Pass
HR (BPM)	Pass
Visual	Pass
Function	Pass
	+0.00

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Pulse Oximeter

X X X X X X	Test Number	49
	Tested on	X
	Tested by	X

<i>Item</i>	Pulse Oximeter
<i>Manufacturer</i>	Nonin
<i>Model</i>	Onyx
<i>Serial Number</i>	500688692
<i>Type</i>	Fingerclip
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result
%SpO2	Pass
HR (BPM)	Pass
Visual	Pass
Function	Pass
	+0.00

General Notes

Pass / Fail

Pass

Signature





1A Carrakeel Drive
 Maydown Industrial Estate
 MAYDOWN BT47 6UQ
 Tel: 028 7134 5188
 Email: info@ebrington-ni.com

TEST CERTIFICATE

Pulse Oximeter

X X X X X X	Test Number	50
	Tested on	X
	Tested by	X

<i>Item</i>	Pulse Oximeter
<i>Manufacturer</i>	Nonin
<i>Model</i>	Onyx
<i>Serial Number</i>	505197632
<i>Type</i>	Fingerclip
<i>Tolerance</i>	
<i>Location/Item Nr</i>	


Test	Result
%SpO2	Pass
HR (BPM)	Pass
Visual	Pass
Function	Pass
	+0.00

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Pulse Oximeter

X X X X X X	Test Number	51
	Tested on	X
	Tested by	X

<i>Item</i>	Pulse Oximeter
<i>Manufacturer</i>	Nonin
<i>Model</i>	Onyx
<i>Serial Number</i>	502323430
<i>Type</i>	Fingerclip
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result
%SpO2	Fail
HR (BPM)	Fail
Visual	Fail
Function	Fail
	+0.00

General Notes Hinge broken leading to sensor failure

Pass / Fail **Fail**

Signature 

TEST CERTIFICATE

Pulse Oximeter

X X X X X X	Test Number	52
	Tested on	X
	Tested by	X

<i>Item</i>	Pulse Oximeter
<i>Manufacturer</i>	Chocimed
<i>Model</i>	MD300
<i>Serial Number</i>	131137500391
<i>Type</i>	Fingerclip
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result
%SpO2	Pass
HR (BPM)	Pass
Visual	Pass
Function	Pass
	+0.00

General Notes

Pass / Fail

Pass

Signature





1A Carrakeel Drive
 Maydown Industrial Estate
 MAYDOWN BT47 6UQ
 Tel: 028 7134 5188
 Email: info@ebrington-ni.com

TEST CERTIFICATE

Pulse Oximeter

X X X X X X	Test Number	53
	Tested on	X
	Tested by	X

<i>Item</i>	Pulse Oximeter
<i>Manufacturer</i>	
<i>Model</i>	E-380
<i>Serial Number</i>	F380XKZ01581
<i>Type</i>	Hand held
<i>Tolerance</i>	
<i>Location/Item Nr</i>	


Test	Result
%SpO2	Pass
HR (BPM)	Pass
Visual	Pass
Function	Pass
	+0.00

General Notes

Pass / Fail

Pass

Signature



TEST CERTIFICATE

Pulse Oximeter

X X X X X X	Test Number	54
	Tested on	X
	Tested by	X

<i>Item</i>	Pulse Oximeter
<i>Manufacturer</i>	
<i>Model</i>	E-380
<i>Serial Number</i>	F380A2491226
<i>Type</i>	Hand held
<i>Tolerance</i>	
<i>Location/Item Nr</i>	

Test	Result
%SpO2	Pass
HR (BPM)	Pass
Visual	Pass
Function	Pass
	+0.00

General Notes

Pass / Fail

Pass

Signature

