



The Experts in  
Medical Refrigeration

---

## USER GUIDES

How to retrieve min and max temperature, viewing and exporting data using the app



---

**Ebrington**  
MEDICAL

**1** OPEN LEC MEDICAL APP TO CONNECT TO DEVICE

**2** VIEWING MIN & MAX TEMPERATURES



The Min/Max temperatures displayed are the lowest and highest recorded AIR and LOAD temperatures since the latest reset.

The date and time of the last reset is displayed here. After resetting the current AIR and LOAD temperatures will be displayed.

**3** RESETTING MIN & MAX TEMPERATURES



**1** To reset the min & max temperatures simply press the reset button

**2** Both AIR and LOAD temperatures will be reset


**NOTE:** After pressing RESET the Min/Max temperatures are stored in the controller for up to 3 months on FIFO bases.

They can be retrieved and downloaded at any time.


## 4 VIEWING TEMPERATURE DATA



1 To view the data select the graph icon

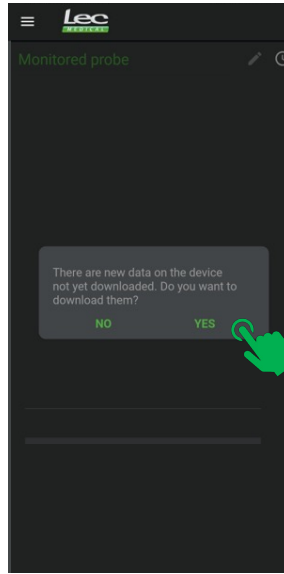


2 When this message appears, select YES

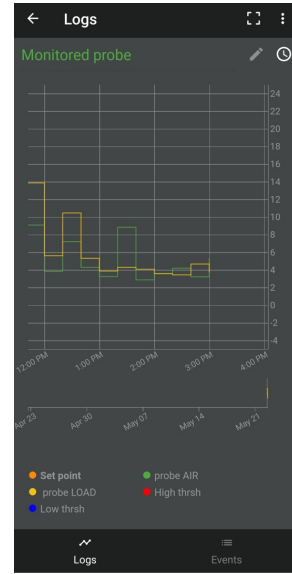


3 When loading screen is complete, temperature data will appear.


## 5 EXPORTING TEMPERATURE DATA

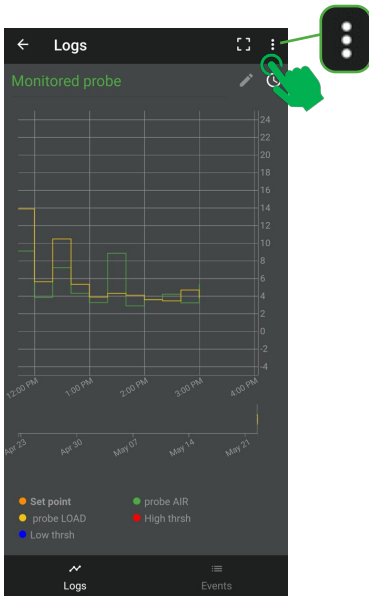


1 To export the data select the menu button top right of the app

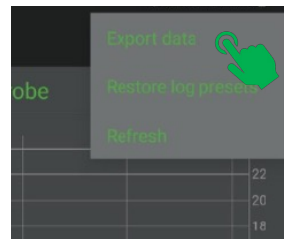

2 Select Export data

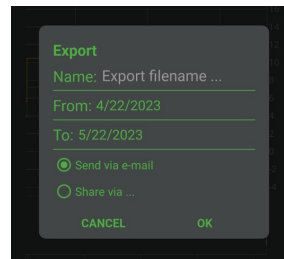

3 Select send via email and then OK

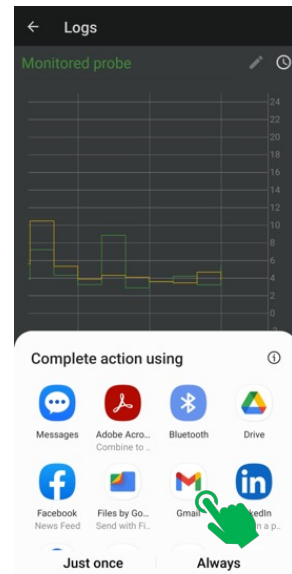



4 Complete action by selecting your preferred email account


## 6 EXAMPLE OF A TEMPERATURE DATA REPORT

Here is an example of the temperature data report you will receive via email:

	A	B	C	D
2152	12/8/2022 7:20:00 AM			
2153	12/8/2022 7:40:00 AM			
2154	12/8/2022 8:00:00 AM			
2155	12/8/2022 8:20:00 AM			
2156	12/8/2022 8:40:00 AM	3.9	4.4	
2157	12/8/2022 9:00:00 AM	4.4	4.8	
2158	12/8/2022 9:20:00 AM	5.8	4.8	
2159	12/8/2022 9:40:00 AM	4.7	4.9	
2160	12/8/2022 10:00:00 AM	3.3	4.9	
2161	12/8/2022 10:20:00 AM	5.9	4.8	
2162	12/8/2022 10:40:00 AM	5.3	4.8	
2163	12/8/2022 11:00:00 AM	4.7	4.8	
2164	12/8/2022 11:20:00 AM	4.0	4.8	
2165	12/8/2022 11:40:00 AM	3.3	4.9	
2166	12/8/2022 12:00:00 PM	5.1	4.8	
2167	12/8/2022 12:20:00 PM	5.9	4.8	
2168	12/8/2022 12:40:00 PM	5.7	4.8	
2169	12/8/2022 1:00:00 PM	5.4	4.8	
2170	12/8/2022 1:20:00 PM	5.2	4.8	
2171	12/8/2022 1:40:00 PM	5.0	4.8	
2172	12/8/2022 2:00:00 PM	4.8	4.8	
2173	12/8/2022 2:20:00 PM	4.7	4.8	
2174	12/8/2022 2:40:00 PM	4.6	4.8	
2175	12/8/2022 3:00:00 PM	4.4	4.8	
2176	12/8/2022 3:20:00 PM	4.3	4.8	
2177	12/8/2022 3:40:00 PM	4.1	4.8	
2178	12/8/2022 4:00:00 PM	3.9	4.8	

## 7 DISCONNECTING FROM THE APP

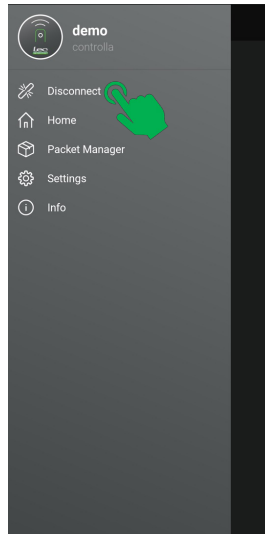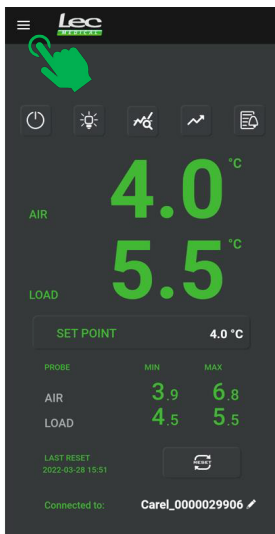

1 Press the **Menu** button 

2 Select **Disconnect**