



The Experts in
Medical Refrigeration

USER GUIDES

How to turn the
appliance off and on
using the controller



1 UNLOCK THE CONTROLLER

Before using any function on the controller.
You will need to unlock the controller...



- 1 Press and hold the navigation button for 3 seconds




- 2 Hold until all dashes appear and then the main screen appears

2 TURNING OFF THE APPLIANCE



- 1 After unlocking the controller, press the **POWER** icon to turn **OFF** the appliance.

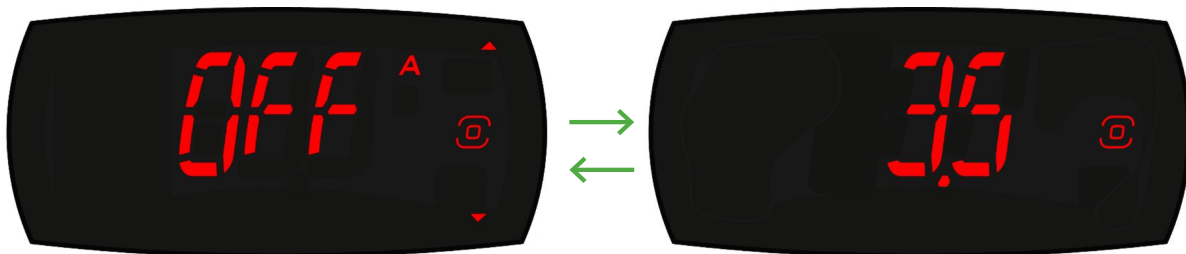

- 2 When the appliance is **OFF** the screen display will alternate between **OFF** and the current **AIR/LOAD** temperature, as above.

2 TURNING ON THE APPLIANCE



- 1 If the appliance is switched off, unlock the controller and press the **POWER** icon to turn **ON** - The screen display will show **ON**.

The current temperature will now be displayed